nosiboo^{*}pro[®]

electric nasal aspirator

reddot award 2014
winner

Safety instructions

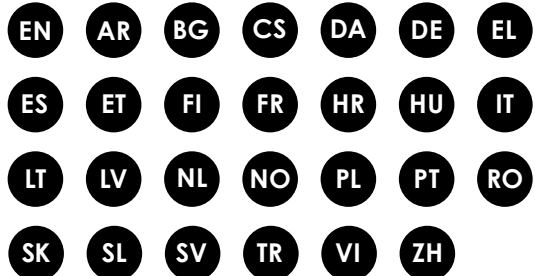
Table of contents

EN Safety instructions	FR Consignes de sécurité	RO Instrucțiuni de siguranță
AR تعليمات السلامة	HR Sigurnosne upute	SK Bezpečnostné pokyny
BG Инструкции за безопасност	HU Biztonsági utasítások	SL Varnostna navodila
CS Bezpečnostní pokyny	IT Istruzioni di sicurezza	SV Säkerhetsinstruktioner
DA Sikkerhedsanvisninger	LT Saugumo nurodymai	TR Güvenlik Talimatları
DE Sicherheitshinweise	LV Norādījumi par drošību	VI Chỉ dẫn an toàn
EL Οδηγίες ασφαλείας	NL Veiligheidsinstructies	ZH 安全指示
ES Instrucciones de seguridad	NO Sikkerhetsinstruksjoner	
ET Ohutusjuhised	PL Instrukcje dotyczące bezpieczeństwa	
FI Turvallisuusohjeet	PT Instruções de segurança	

^{*}nosiboo[®]pro

electric nasal aspirator

EN

Safety instructions

Dear Customer,

thank you for purchasing our Nosiboo Pro Electric Nasal Aspirator.

Please, follow the instructions provided for your device. To obtain the very best performance for your nasal aspirator, please read the **Quick Guide** too. For more information we recommend reading the **Instructions for Use**:

ATTRACT Kft. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

EUDAMED SRN: HU-MF-000004252

CE 2409

Please keep these documents for future reference.

Please read all instructions carefully before using the device.

The warranty will be only validated if the device was used according to its intended purpose and does not bear signs of damage.

Nosiboo Pro electric nasal aspirator is a medical device. You may exercise your right of withdrawal only if the device has not been used, is in undamaged condition, and the hygienic packaging of the device, the Colibri head, and the nose tip remains unopened. In all other cases, the purchase price cannot be refunded.

Symbols

Class IIa medical device

Read the Instructions for Use

GTIN – Global Trade Item Number (UDI-DI)

Lot number

Serial number

Model number (catalogue number)

Unique Device Identifier

Protection against solid particles of diameter greater than 12.5 mm; protected from condensation.

Double insulation

Name and address of the manufacturer

Keep out of reach of children

Keep away from sunlight

Fragile – Handle with care

Keep dry

Collect separately the electrical and electronic equipment waste!

Caution!

Intended use of the device

The Nosiboo Pro electric nasal aspirator is a medical device intended for use by laypersons. It is designed for use at home to remove nasal mucus from children primarily under the age of 6. Its purpose is to help manage symptoms of cold and nasal congestion, and to prevent sinus infections in children.

General use

The device cannot be used in oxygen-rich environments and in clinical circumstances! The medical device is for home use only.

Use only as directed. Misuse can be dangerous.

The device should only be used by adults.

Do not use the nasal aspirator for nasal washing! Aspirating water with the electric device will result in loss of warranty.

In case of unexpected failure or malfunction, immediately switch off the device.

Do not allow children to play with the device, it is NOT A TOY. (E.g., the power cable can wind easily around a child's neck.)

Do not leave the device unattended near children.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the power cable and the metal parts of the device with wet hands. Do not cover the vents on the back of the device while in use.

The device is intended to be used only for removing nasal mucus from a child's nose. It shall not be used for any other purposes.

Keep the device and its accessories out of reach of children.

Make sure that all parts are properly attached before using the device.

Do not use the device without the air mesh.

Make sure that the housing, the connecting tube, and the Colibri head are not damaged or cracked.

Use the device only with the Colibri head supplied by the manufacturer.

The use of non-original spare parts poses technical and medical risk for which the manufacturer accepts no responsibility! The use of non-original spare parts may cause malfunction or damage to the device and will result in loss of warranty. Only use official Nosiboo parts!

The part of the Colibri head attached to the connecting tube shall point downwards while in use.

Do not turn the Colibri head upside down meanwhile in use.

Cleaning, maintenance and storage

Keep the device away from heat (e.g. furnace, heater, or convector) because it can cause damage to the insulation.

Do not tie knots on the power cable or the connecting tube. To avoid damage, do not place heavy items on the cable.

A damaged power cable may cause electric shock. Do not use the power cable and the device if they are damaged or cracked.

Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes. It may be life-threatening.

When you disassemble the Colibri head for cleaning, do not leave the parts within reach of children. Children may chew or swallow the small parts.

It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!

Do not wash the housing or the power cable with water or any other liquid. Do not handle the device with wet hands. This may cause electric shock or malfunction.

Always unplug the power cable from the outlet after using the device.

The device should be stored in a cool, dry place. Avoid direct sunlight or heat.

Power source

Use the device only with the power cable supplied by the manufacturer. Failure to do so may result in fire hazard.

Be sure the plug of the power cable is compatible with your power outlet!

The device should be connected to or disconnected from the power network only if the control button is in OFF position (the device is switched off).

Important information about the device

Nosiboo Pro devices are equipped with an internal power supply unit that generates low voltage.

Disassembling the device housing is prohibited and life-threatening.

Specifications

Max. motor performance: 500 W
Adjustable motor performance: 150-390 W
Performance: 420 VA
Differential pressure: 0.04-0.08 bar
Max. air flow: 30-42 l/min
Power supply: 230 V, 50 Hz
IP class: IP21
Protection class: II
Operation type: no continuous operation
Operation length: on 15 min/appr. off 50 min
Duration of continuous use: max 15 min.
Safety switch release time: after 15 minutes
Tripping temperature of the thermal switch: 95 °C.
Average housing temperature: 30-35 °C after 1 min continuous use
Noise level: 60-70 dB
Expected life of the housing (electrical parts): 3 years
Expected life of the Colibri head: 3 months
Diameter of the nasal insertion end of the standard nose tip:
max. 7.0 mm
Length of the nasal insertion end of the standard nose tip: max.
8.0 mm
Diameter of the nasal insertion end of the newborn's nose tip:
max. 4.9 mm
Length of the nasal insertion end of the newborn's nose tip:
max. 12.6 mm
Weight: 1090 g
Dimensions (housing, W x H x D): 198x162x172 mm
Dimensions (complete package): 180 x 180 x 225 mm
Operating conditions:
1. Temperature range: +5 °C to +40 °C
2. Relative humidity: max. 75%
3. Atmospheric pressure / barometric pressure range: 525-795 mmHg / 70-106 kPa
Transportation conditions:
1. Temperature range: -20 °C to +65 °C
2. Relative humidity: 10-75%
3. Atmospheric pressure / barometric pressure range: 150-795 mmHg / 20-106 kPa
Storage conditions:
1. Temperature range: 0 °C to +40 °C
2. Relative humidity: 10-75%
3. Atmospheric pressure / barometric pressure range: 525-795 mmHg / 70-106 kPa
MD Category: II.a
MDD/MDR: MDD Legacy Device
EUDAMED SRN: HU-MF-000004252

We reserve the right to make technical changes.

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

Contact

Contact details of the manufacturer and responsible person
ATTRACT Kft.
H-7622 Pécs, Siklósi út 1/1., Hungary
Tel.: +36 72 551 642 (Mo-Fr: 8 am – 4 pm)
E-mail: customerservice@nosiboo.com

PRRC (Person Responsible for Regulatory Compliance):
ATTRACT Kft. – Quality Manager
H-7622 Pécs, Siklósi út 1/1., Hungary
Tel.: +36 72 551 642 (Mo-Fr: 8 am – 4 pm)
E-mail: quality@nosiboo.com

For more information we suggest reading the Instructions for Use and our Quick Guide as well.

Warranty

The warranty period provided by the manufacturer for the device from the date of purchase is: 24 months.

The warranty does not apply to products used improperly, and the manufacturer cannot be held liable for any damage resulting from improper use.

For products placed on the market in the European Union:
Please keep the warranty card and the purchase invoice for 24 months for warranty purposes. The warranty card included in the device packaging is valid only together with the invoice.

For further details, please refer to the online Instructions for Use.

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