Instructions for Use

Nosiboo Eco – manual nasal aspirator

EN Manufacturer **AR** المصنع **BG** Производител **CS** Výrobce
DA Producent **DE** Hersteller **EL** Κατασκευαστής **ES** Fabricante
ET Tootja **FI** Valmistaja **FR** Fabricant **HR** Proizvođač
HU Gyártó **IT** Produttore **JA** 製造メーカー **KO** 제조업체 **LT** Gamintojas
LV Ražotājs **NL** Fabrikant **NO** Produsent **PL** Producent **PT** Fabricante
RO Producător **SK** Výrobca **SL** Proizvajalec **SR** Произвођач
SV Tillverkare **TR** Üretici **VI** Nhà sản xuất **ZH** 制造商

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EN

1. *Intended use of the device:*

The Nosiboo Eco manual nasal aspirator is a medical device intended for use by laypersons. It is designed for use at home to remove nasal mucus from children primarily under the age of 6. Its purpose is to help manage symptoms of cold and nasal congestion, and to prevent sinus infections in children.

2. *Assembly of the device:*

Use original Nosiboo parts! Our nasal aspirators work effectively only with original Nosiboo parts/accessories.

1. Clean the parts of the Colibri head (see chapter "Cleaning and storage of the device").
2. Insert the collared tube into the Colibri head part with connector.
3. Attach it to the Colibri head part without connector.
4. Push the nose tip into place.
5. Connect one end of the tube to the connecting point of the Colibri head.
6. Connect the other end of the tube to the mouthpiece.

3. *How to use the device:*

The device should only be operated by an adult. Nasal mucus is removed from the child's nose by vacuum and airflow generated through the user's own lung power. The aspirated secretion travels from the child's nose through the nose tip into the Colibri head. Thanks to its patented design, the Colibri head collects the mucus without a separate filter element and is easy to clean after use.

Use the device with the child lying down during infancy, and in a stable sitting position from little childhood. Place the mouthpiece in your mouth and gently insert the nose tip into the child's nostril, slightly angled toward the sinus. While holding it in place, gently move the nose tip as needed and apply intermittent suction through the tube using your mouth. During use, the connection point between the tube and the Colibri head should face downward. If needed, the device can be used several times a day—for example, after waking up, before feeding, and before sleep—to support calm breathing. Isotonic saline solutions may help dissolve dried or thick nasal secretions. After applying the solution, wait a few minutes before using the device. Nosiboo Eco can hold up to 5 ml of diluted liquid.

4. Cleaning and storage of the device:

Always clean the Colibri head after each use! Clean the tube only if any secretions have accidentally entered it. To clean, detach the Colibri head from the tube along with the mouthpiece. Disassemble the Colibri head, then wash all parts and the mouthpiece with lukewarm water and dishwashing liquid (use the cleaning brush to clean the nose tip). Alternatively, you may soak the parts in a liquid disinfectant intended for baby products, following the manufacturer's instructions. Let all components dry completely. It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat! Store the device in a closed space, and do not expose it to direct sunlight or heat!

5. Warnings:

Avoid the following dangerous situations to prevent potentially serious injuries or fatal accidents. Do not leave the dismantled Nosiboo Eco parts within reach of children, as they may put small components in their mouth and swallow them. Never leave the device unattended near children or individuals with reduced physical or mental capacity. Do not allow children to play with the device. IT IS NOT A TOY! Before each use, ensure that all components are properly assembled. If the child develops an allergic rash or redness around the nose after using the device, discontinue use and consult a doctor. Do not use the device in cases of haemophilia, frequent nosebleeds, congenital malformations (e.g., cleft palate), or while using blood-thinning medication.

6. Waste disposal:

If free from contamination, the device can be disposed of through selective waste collection.

7. Specifications:

Item number: ECO-01-03, GTIN (UDI-DI): 05999885976874

Classification: Class I medical device

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